mikrobiologie labor technik

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PRINCIPIU

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
Enriched Chocolate Agar

PRINCIPLE

This medium is recommended for the growth of fastidious microorganisms such *Neisseria* and *Haemophilus*. Hemoglobin provides hemin (or X factor). Thermosensitive V factor (coenzyme NAD) is added after sterilization when incorporating polyvitamin supplement. Chocolate enriched agar is adapted for the analysis of biological samples such as CRL and blood cultures.

FORMULA

Components	g/l
Peptocomplex	15.00
Corn starch	1.00
Sodium chloride	5.00
Dipotassium Hydrogen phosphate	4.00
Potassium dihydrogen Phosphate	1.00
Cooked horse blood	100.00 ml
Polyvitamin supplement q.s.p.	1 l
Agar	12.00
Final pH: 7.2 ± 0.2 at 25°C	

METHOD

Melt the chocolate base and cool to 42 ± 1°C and add 5 to 10% of blood. Under slow constant homogenization heat for 5 minutes at 70°C (to cook the blood). Cool for a second time and pour in sterile plates.

PROCEDURE

Warm the plates to 37°C prior to inoculation. Incubate 24 to 48 hours under humid and possibly CO₂ enriched atmosphere.

RESULTS

Neisseria gonorrhoeae colonies grow in 24 hours. They are small, greyish with regular

contours and 0.5 to 1 mm Ø. If the incubation is prolonged these colonies grow and become irregular.

Neisseria meningitidis grow faster : they are slightly bigger with regular contours and with greyish and transparent coloration. Colonies can also be mucous.

Neisseria identification will be followed by a Gram coloration, oxydase test and a biochemical identification systems.

Haemophilus species grow easily on enriched chocolate medium, specially when the atmosphere contain CO₂. Colonies are often mucous, translucent and greyish with a diameter of 0.5 to 1 mm. Species diagnostic will be done by testing X & V factor requirements and by using a biochemical rapid identification systems.

BIBLIOGRAPHY

1. Thayer J.D. and Martin J.E. 1966. Improved Medium for Cultivation of *N. gonorrhoeae* and *N. meningitidis*. Public Health Rep. 81:559-562.
2. Riou J.Y. et Guibourdenche M. 1992. Méthodes de laboratoires *Neisseria* et *Branhamella*. Inst. Past. – pp30-38.
3. COFRAC. Programme 116/00/BA24/00

PACKAGING

Dehydrated medium

Store between 1 and 30°C
140119A: Flask of 500 g

Enriched chocolate, ready to use plate

Store between 2 and 8°C
110119: Pack of 20 plates 90 mm Ø
120177: Pack of 32 tubes of 7 ml